


FitMax™ CPAP Total Face Mask - Fitting guide

Size guide



1. Select the correct size using the size guides that are supplied in the outer box.

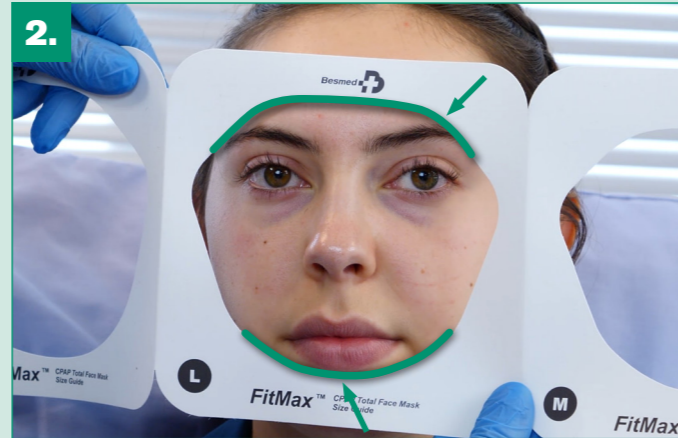

2. The top of the mask seal should contact the face just above the eyebrows and the bottom just below the lower lip.

Fitting the mask



3. Attach the elbow to the swivel connector and check its smooth rotation.



4. If the orange elbow is used - Check the vent holes on the elbow and ensure that the anti-asphyxiation valve assembly is present and undamaged.



5. Lift the headstrap over the patient's head and position the mask holding it from the faceplate protrusion, never from the elbow.



6. Clip the headstrap into place.



7. Adjust the headgear using the side, bottom and top straps. **The mask should sit gently on the face and not feel tight.**



8. **The top strap is adjusted to keep the side straps away from the patient's ears.**



9. Start the gas flow and attach the breathing system using a push and twist action. **Make final adjustments while the patient lies down.**



10. **Ensure that the headgear is not overtightened.** Straps should be adjusted to minimise contact pressure while ensuring a good seal.



FitMax™ CPAP Total Face Mask

The FitMax CPAP total face mask is a single patient use NIV interface for use with CPAP and bilevel treatment within the hospital environment. It is available in three adult (>30kg) sizes and two elbow options are provided (**Vented** with anti-asphyxia valve and **Standard** non-vented).



Code	Description	Colour coded elbow	Box Qty.
For use with single limb breathing systems			
2290020	FitMax CPAP total face mask, with vented anti-asphyxia elbow, head strap, medium	Orange	10
2290030	FitMax CPAP total face mask, with vented anti-asphyxia elbow, head strap, large	Orange	10
2290040	FitMax CPAP total face mask, with vented anti-asphyxia elbow, head strap, extra large	Orange	5

Code	Description	Colour coded elbow	Box Qty.
For use with dual limb breathing systems			
2290070	FitMax CPAP total face mask, with standard elbow, head strap, medium	Blue	10
2290080	FitMax CPAP total face mask, with standard elbow, head strap, large	Blue	10
2290090	FitMax CPAP total face mask, with standard elbow, head strap, extra large	Blue	5

To view our full patient interface solutions visit: www.intersurgical.com/info/patient-interfaces

The FitMax masks are manufactured by Besmed Health Business Corporation, Taiwan and distributed by Intersurgical Ltd