




StarMed Ventukit

Hood for CPAP therapy with built-in Venturi flow driver

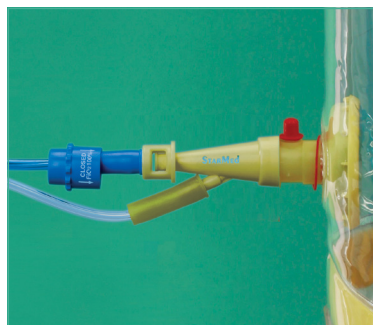

Critical Care ▪ Patient Interfaces



StarMed Ventukit hood for CPAP therapy with built-in Venturi flow driver

The Ventukit is a complete and easy to use system simply activated by connecting to the O₂ source available in the hospital departments. StarMed Ventukit unique design consists of oxygen tubes and a Venturi flow driver integral to a CPAP hood. The kit also includes an adjustable PEEP valve for immediate therapy activation.


Read more evidence on www.intersurgical.com.au

Technical Features

- The integrated Venturi flow driver can generate high patient flows while consuming a reduced amount of compressed oxygen
- O₂% can be adjusted from 40% to 100% at each PEEP level
- The adjustable PEEP valve can provide patient pressures in a range of settings up to 20 cmH₂O
- The user is guided by a performance table integral to the

Features and benefits

- Transparent for ease of patient monitoring
- Integral Venturi flow driver with oxygen supply tubes
- Patient access port with bi-directional anti-asphyxiation valve
- Integrated pressure manometer
- Comfortable underarm fastening



Dual flow meter
Code: STS.15/30
Dual oxygen flow meter 15 + 30 L/min with O₂ coupling

Requirement		Setting	Flow delivered
O ₂ % approx	PEEP cmH ₂ O	Flow O ₂ L/min A + B	Flow approx L/min
40%	5	9 + 0	46
	7.5	9 + 0	47
	10	10 + 0	48
	12.5	10 + 0	46
	15	11 + 0	47
50%	20	12 + 0	48
	5	8 + 9	47
	7.5	8 + 9	47
	10	9 + 9	45
	12.5	9 + 8	48
	15	10 + 8	50
	20	12 + 9	54

Requirement		Setting	Flow delivered
O ₂ % approx	PEEP cmH ₂ O	Flow O ₂ L/min A + B	Flow approx L/min
60%	5	7 + 16	80
	7.5	8 + 16	77
	10	8 + 14	72
	12.5	9 + 13	65
	15	9 + 13	60
100%	20	10 + 14	60
	Feed with a minimum O ₂ flow of 40 L/min		

Ventukit CPAP hood with Venturi flow driver, bi-directional anti-asphyxiation valve and patient access port, adjustable PEEP valve, ear plugs and measuring tape

Code	Size	Neck circumference	Box Qty.
CV100XXL/2W	Extra Extra Large	≥50cm	5
CV100XL/2W	Extra Large	45-52cm	5
CV100L/2W	Large	40-47cm	5
CV100M/2W	Medium	34-41cm	5
CV100S/2W	Small	27-34cm	5
CV100XS/2W	Extra Small	17-27cm	5

 www.intersurgical.com.au/info/starmed

[Make an enquiry](#)

IS1.21_AU • Issue 5 06.16



Unit 4, 151 Beauchamp Road, Matraville, NSW, 2036
T: +61 (0) 2 8048 3300 info@intersurgical.com.au www.intersurgical.com.au

UK • Ireland • France • Germany • Spain • Portugal • Italy
Netherlands • Sweden • Denmark • Lithuania • Russia
Czech Republic • Turkey • South Africa • China • Japan
Taiwan • Philippines • USA • Colombia • Australia



Intersurgical Ltd is certified to ISO 14001, ISO 9001 and ISO 13485

Please think before you print
Save energy and paper.
If you must print this e-mail please print it double sided